


## Maximum Single Use Guidance

- Concentration of lidocaine HCl in product = 0.4 mg/cm<sup>2</sup> [1]
- Maximum single dose of lidocaine HCl within 24-hr timeframe = 300 mg or 4.5 mg/kg [1]

**Please Note:** Pelashield™ PainGuard™ does not need to be removed after application and can be reapplied according to patient care protocol (such as once a week or every 3-4 days). The chart below shows the maximum number of pieces (by body weight) that may be applied in a single application over a 24-hour period. These limits are primarily relevant for patients at lower body weights or when applying larger-sized pieces of PainGuard™.

**Pediatric Use Disclaimer:** The following values are derived from adult dosing calculations and extrapolated to lower body weights for reference purposes only and should not be interpreted as strict pediatric dosing recommendations. This guidance is not intended to replace clinical decision-making; dosing for pediatric patients should be determined using clinical judgment and established institutional protocols.

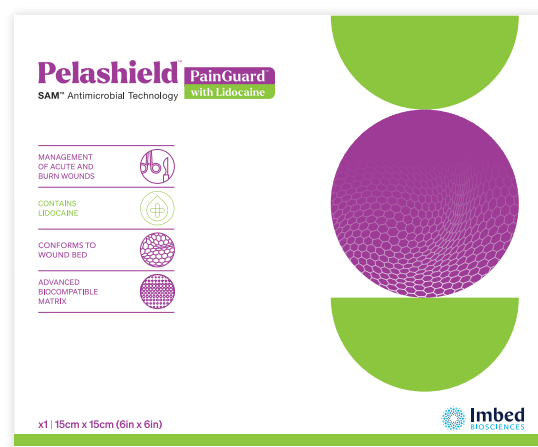

| Patient Weight |             | Maximum Number of Pieces per Single Dose |            |
|----------------|-------------|------------------------------------------|------------|
| Pounds         | Kilograms   | 10 x 23 cm                               | 15 x 15 cm |
| 147 and up     | 66.7 and up | 3                                        | 3          |
| 140            | 63.5        | 3                                        | 3          |
| 130            | 59          | 2                                        | 2          |
| 120            | 54.4        | 2                                        | 2          |
| 110            | 49.9        | 2                                        | 2          |
| 100            | 45.4        | 2                                        | 2          |
| 90             | 40.8        | 2                                        | 2          |
| 80             | 36.3        | 1                                        | 1          |
| 70             | 31.8        | 1                                        | 1          |
| 60             | 27.2        | 1                                        | 1          |
| 50             | 22.7        | 1                                        | 1          |
| 45             | 20.4        | 1                                        | 1          |
| 40             | 18.1        | 0                                        | 0          |
| 35             | 15.9        | 0                                        | 0          |
| 30             | 13.6        | 0                                        | 0          |
| 25             | 11.3        | 0                                        | 0          |
| 20             | 9.1         | 0                                        | 0          |
| 15             | 6.8         | 0                                        | 0          |
| 10             | 4.5         | 0                                        | 0          |
| 5              | 2.3         | 0                                        | 0          |